**HUMAN CARE**  
Your Life. Your Way.

# neXus 3

Reconditioning Manual/Rekonditioneringsanvisning/  
Manuel de maintenance

# Content

---

<b>Reconditioning Manual (EN)</b>	<b>3</b>
Maintenance, inspection and testing	3
Warnings and recommendations	5
Exploded view	6
Spare Parts	7
<b>Rekonditioneringsanvisning (SV)</b>	<b>10</b>
Underhåll, inspektion och test	10
Varningar och rekommendationer	12
Sprängskiss	13
Reservdelar	14
<b>Manual de maintenance (FR)</b>	<b>17</b>
Entretien, inspection et essais	17
Avertissements et recommandations	19
Vue éclaté	20
Pièces de rechange	21

## Maintenance, inspection and testing

EN

To ensure that your walking aid is functioning properly and safely, an annual inspection should be done only by an authorized agent. The following maintenance should be done regularly:

### ○ **Cleaning**

Regular cleaning with a mild liquid detergent or wiping with a damp cloth is recommended. **DO NOT** use any toxic cleaning materials, chlorine bleach or abrasive cleansers on the product as it will damage it. **DO NOT** pressure wash.

### ○ **Post treatment**

Lubricate any moving parts. Ensure the rollator is free of residue moisture to avoid rusting its components. **DO NOT** keep near a heat source.

### ○ **Chassis**

Check the weldings for any visible signs of damage. Should damage be evident, cease usage and immediately have the rollator inspected. Replace if necessary.

### ○ **Finish**

The rollator is painted and sealed for a superior finish, however scratches can occur with impact. Scratches in the frame may result in corrosion.

### ○ **Corrosive components**

The rollator does contain components that may rust if left in a moisture setting. Periodically check all welding points and screws. It is recommended that an authorized agent perform this inspection. If corrosion is present,

replace parts. If corrosion is extensive, replace rollator.

### ○ **Periodic inspections**

At minimum, it is recommended that an authorized agent perform an annual inspection of the rollator and its components to ensure it is functioning as intended. Do not use the rollator if any components are damaged or inoperable. Have an authorized agent perform necessary repair and/or replacement.

### ○ **Handle grip**

Periodic replacement of the handle grip may be necessary with usage. Replace as required.

### ○ **Braking system**

The brakes should function in both the control and park position. The handle should stay in the “locked” position when pushed down and easily move up when “squeezed” towards the grip. Should these control functions not operate as intended, cease usage and immediately have the rollator repaired.

### ○ **Height adjustment**

Control function. Check star handle to ensure it is securely fastened.

### ○ **CE-marking**

Ensure that the CE-marking is present and legible.

**Folding mechanism**

Control function

**Wheels**

Ensure the wheels are not worn out and spin easily to ensure proper movement of the rollator. If any noise is heard, have an authorized agent inspect product.

**Seat**

Periodically check to ensure that the seat mechanism and screws are firmly in place. If any components are not secure, do not sit on the rollator and have it inspected for repair immediately. If used, ensure the seat pad is properly mounted in the seat place for maximum comfort.

**Stability**

The wheels should all rest firmly on the ground and that there should be no distortion of the frame.

**Walking Test**

Before using your rollator, take a small walk with it to ensure it is properly functioning.

---

**Authorized Dealer/Service Agent**

**Last Service Date**

## **Intended use**

This rollator was designed to act as a support device while walking or as a resting seat while stationary and locked into position. It should never be used as a transport device. Never attempt to alter the rollator in any way.

## **Note!**

**Reconditioning must be done only by an authorized agent. If incorrectly performed by an unauthorized individual, it could result in an accident/injury.**

To locate an authorized agent, contact your local office:

Europe: [info.se@humancaregroup.com](mailto:info.se@humancaregroup.com)

Canada: [info.ca@humancaregroup.com](mailto:info.ca@humancaregroup.com)

United States: [info.us@humancaregroup.com](mailto:info.us@humancaregroup.com)

Australia: [info.au@humancaregroup.com](mailto:info.au@humancaregroup.com)

Netherlands: [info.nl@humancaregroup.com](mailto:info.nl@humancaregroup.com)

**When exchanging parts on the rollator, use only Human Care authorized parts to ensure continued CE-validation.**

**If corrosion is more than a film of rust, discard and recycle rollator according to local regulations.**

**Never use this rollator in any way other than for its intended purpose.**

**It is strongly recommended that a therapist or doctor assess your needs prior to using this product.**

**Inspect your rollator frequently prior to usage to ensure there is no damage or wear.**

**User must not exceed this product's weight capacity!**


NeXus 3

200kg/441lb

**Note: Reconditioning guidelines may vary from country to country. Ensure that any reconditioning performed follows local regulations.**

# Exploded View

EN

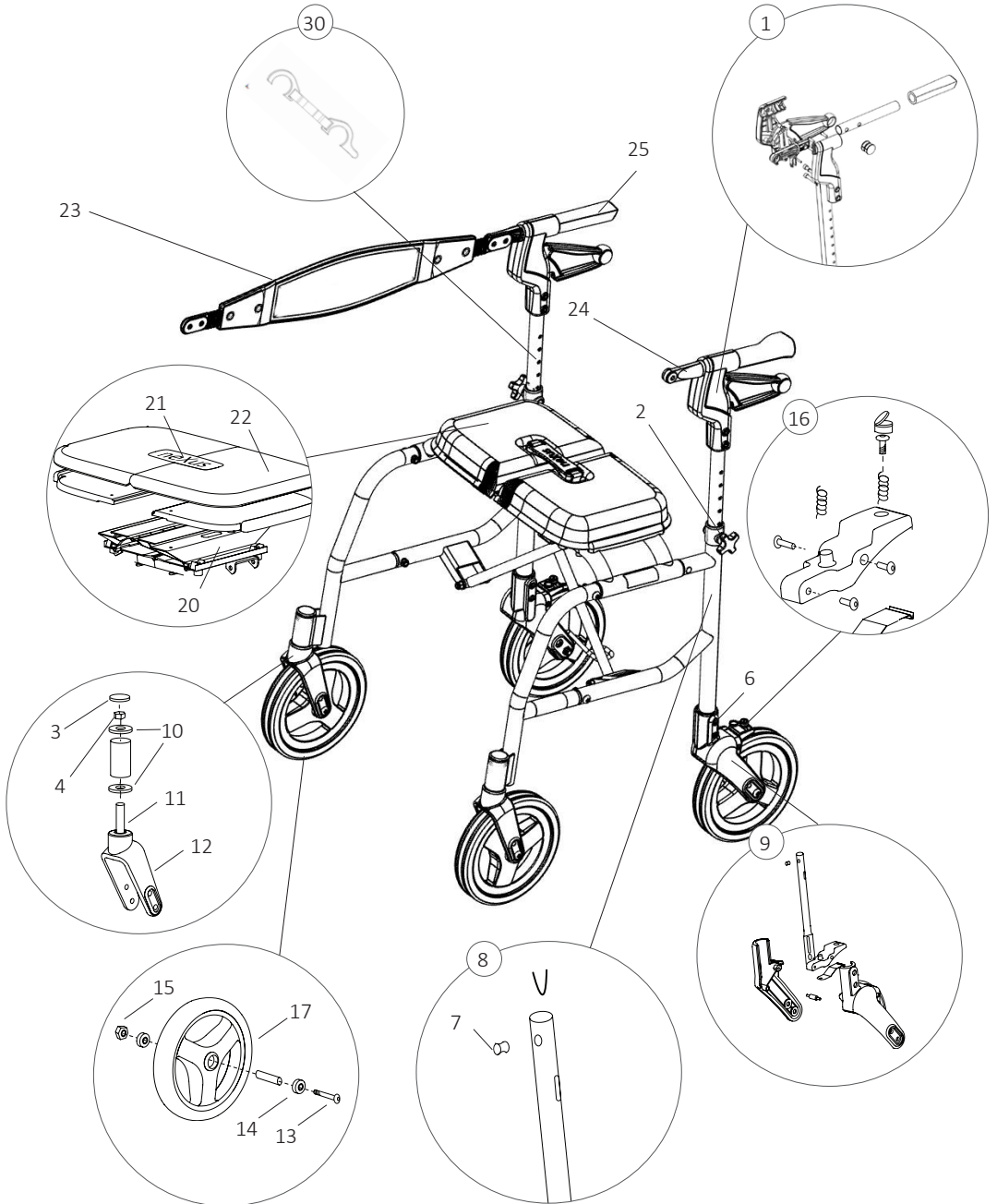No	Designation (Qty/Rollator)	UofM	Art. no	Additional Instructions
1	<b>Brake Handle Complete (1 required/ea)</b> Right - Red Ball Left- Red Ball	EA	93243 93201	Standard with rollator
	<b>Brake Handle Complete Short (1 required)</b> Right - Red Ball Left- Red Ball	EA	93245 93244	Optional
2	<b>Star Handle and Bolt (2 required)</b> Star Handle (Wing Nut) Bolt	EA	89096	For models after 2009
3	Bolt Cap	EA	4823	
4	Front Fork Stem Nut	EA	H48025	
5	Rear Fork Tubing Nut	EA	H48031	
6	Rear Fork Tubing Screw	EA	H48030	
7	Lock Button	EA	4833	
8	<b>Lower Brake Rod Assembly (2 required)</b> Left- Super Low Left- Low Left - Standard Right- Super Low Right- Low Right- Standard	EA	93256 93257 93258 93259 93260 93261	
9	<b>Rear Fork Assembly (2 required)</b> Left- Super Low Left- Low Left - Standard Right- Super Low Right- Low Right- Standard	EA	AS48013 AS48015 AS48017 AS48014 AS48016 AS48018	Includes inner/outer fork, lower brake rod assembly, brake pad complete and cap)
10	Front Fork Bearing (2 required/fork)	EA	H48026	
11	Front Fork Stem Bolt	EA	H48024	
12	Front Fork	EA	4825	
13	Axle Bolt	EA	H48020	
14	Wheel Bearing (2 required/wheel)	EA	H48022	
15	Axle Nut	EA	H48021	
16	<b>Brake Pad Complete (2 required)</b> Hex Adjustment Screw Brake Pad Spring Pivot Sleeve Screws (2) Brass Pivot Screw Adjustment Screw Lever Spring Lever Brass Lug	EA	76011	

17	<b>Wheel Complete, grey (4 required)</b> Wheel Bearings (2/wheel) Bearing Spacer	EA	93282-01	
17	<b>Wheel Complete, black (4 required)</b> Wheel Bearings (2/wheel) Bearing Spacer	EA	93282-02	
18	Plastic Cap Screw Cover (Rear Fork)	EA	H45003	
19	Curb Climber	EA	4879	
20	Folding Seat Mechanism	EA	P45007	
21	<b>Seat Handle</b> Red	EA	P45008	
22	Seat Pad Set (2 pc)	EA	93263	
23	<b>Backstrap Complete</b> Backstrap Backstrap Clamp Screw Backstrap Clamp Nut	EA	90391	
24	<b>Backstrap Clamp</b> Left Right	EA	4880 4881	
25	Handle Grip	EA	20-024	
26	Wire Basket (not shown)	EA	4510	
27	Replacement Soft Bag Bracket Kit	EA	90276	Included with soft bag
28	Soft Zippered Bag Complete	EA	93280	Standard with rollator
29	Soft Zippered Bag- 5 Pack	EA	93281	
30	Locking Snap		73004	

<b>For 4600 models:</b>				
No	Designation (Qty/Rollator)	UofM	Art. no	Additional Instructions
	<b>Brake Handle Complete Right (1 required)</b> Regular Tall	EA	P48535 P48555	
	<b>Brake Handle Complete Left (1 required)</b> Regular Tall	EA	P48545 P48565	
	<b>Lower Brake Rod Assembly (2 required)</b> Left- Super Low Left- Low Left - Standard Right- Super Low Right- Low Right- Standard	EA	76013 76016 76018 76015 76017 76019	
	Soft Bag	EA	80501Q5	Only sold in 5 pack

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