Drehscheibe Turning Disc

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Table of Contents

1 General Information

- 1.1 Product versions
- 1.2 Accessories
- 1.3 Scope of delivery
- 1.4 Storage, transport
- 1.5 Service life / lifecycle

2 Safety information

- 2.1 Symbols used
- 2.2 Function check

3 Using the product

- 3.1 Indication and further use
- 3.2 Contraindications
- 3.3 Application risks
- 3.4 Areas of application
- 3.5 Application illustration
- 3.6 Application
- 3.7 Approved accessories, combination with other products

4 Product marking / CE

5 Cleaning, care and disinfection

6 Technical Data

7 Warranty

8 Disposal information

1 General Information

Thank you for choosing a Petermann product.

Please read the user manual carefully before using the product for the first time!

The current understanding in modern occupational health and safety is that people in nursing professions experience significant work-related physical strain. Back injuries are common. There are many situations in which a patient has to be transferred from a bed, a wheeled stretcher, ambulance stretcher, an X-Ray table or a wheelchair to a different stretcher or seat. The strain on the user and the patient / care-dependent can be reduced significantly through the use of transfer aids.

The use of transfer aids should be conditional upon the user having learned to use the equipment properly. The user has to decide for themselves what equipment is suitable to the patient's situation and the intended sequence of movements.

The Petermann Turning Disc is part of a product line which is used specifically to aid in the transfer of patients in hospitals and other health care facilities, as well as in home care.

The turntable can be used outdoors and in the bathroom. However, care must be taken to ensure that it is placed on a **level and dry surface**, otherwise it cannot be transferred safely!

The product may only be used for the specified purpose and under the specified conditions.

The information in this user manual is for the patient who is being transferred and moved, as well as the user performing the transfer / position change.

Contact us if you require additional information or assistance using the Petermann product.

1.1 Product versions

Item no.: PM-5080 Turning Disc

1.2 Accessories

No accessories are available for this device.

1.3 Scope of delivery

Turning Disc
User manual

1.4 Storage, transport

Store/transport at -5°C – 40°C
Other: protect from sunlight

1.5 Service life / lifecycle

If the product is used in accordance with the safety instructions, then the expected product service life is **five years**. However, the effective service life may vary depending on the frequency and intensity of use and the number of cleaning and disinfection cycles.

Perform regular function checks on the product and be sure to observe the safety information (see item 2).

2 Safety information

WARNING indicates a potential danger which could lead to personal injury or material damage.

- It is the responsibility of the organisation / the user to determine, on the basis of the circumstances (weight / health condition of the patient) and the capabilities of the user, whether several users may be required for the transfer and/or whether additional safety measures are required.
- Defective products may no longer be used.
- Ensure that the transfers are performed by the users in a back-friendly manner.
- Weight limits must be observed (200 kg)
- Ensure that the patient's feet are placed correctly and completely on the Turning Disc and that the Turning Disc is resting securely on the floor.
- Ensure that the Turning Disc isn't an obstacle in the room and that nobody can stumble over it.
- Check the transfer surfaces carefully to ensure they are fixed in place securely (e.g. wheelchair, bed).
- Transfer products are subject to a natural ageing process. Inspect the equipment regularly to ensure proper functionality

2.1 Symbols used

| | | | |
|---|-------------------------|---|-------------------------------|
|  | Wipe off |  | max. load capacity see item 6 |
|  | Do not bleach |  | Manufacturer |
|  | Do not tumble |  | Batch no. / lot no. |
|  | Do not iron |  | Identification code |
|  | Do not dry clean |  | Conformity mark |
|  | Warning |  | Item number |
|  | Observe the user manual |  | Medical device label |

2.2 Function check

Transfer products are also subject to a natural ageing process. Inspect the equipment regularly and carefully, in particular before each use.

| | |
|---|--|
|  <p>Visual inspection Ensure that material, seams, handle loops and buckles are undamaged and in functional order.</p> |  <p>Stability test Put pressure on to test stability</p> |
|  <p>Sliding properties Place the product on a firm surface Pull your hands over the product with pressure Good glide - easy gliding Low glide: no gliding/moving possible</p> | |
|  <p>➤ Defective products may no longer be used!</p> | |

3 Using the product

3.1 Indication and further use

Due to the wide range of possible applications, the indications cannot be tied to a specific clinical picture, since the Turning Disc is suitable for all applications involving people who require assistance during transfers. The product is used for:

- Joint friendly transfer for the patient
- Improve safety during transfer
- Facilitate the transfer in tight spaces or on small areas.
- Transfer with mobilisation to standing, is also possible with the Turning Disc for patients with high muscle extension tone

This product may be reused.

Before reusing the product, be sure to perform a thorough function check (see item 2.2) as well as cleaning or disinfection of the product. (see item 5)

3.2 Contraindications

Contraindications cannot be tied to a specific clinical picture. Under consideration of the safety information (see item 2), the product may not be used if the patient has any of the following conditions:

- All diagnoses where the patient is not permitted to apply any weight to the lower extremities
- Low stability in both knee joints → only possible with leg belt! (PM-3005)

3.3 Application risks

- User not standing firmly → Risk of falling
- The movement is performed surprisingly or too quickly → risk of falling and impact particularly in the case of anxious patients, attention must be paid to the patient and his condition
- Patient has insufficient stability → Risk of falling

3.4 Areas of application

The product can be used in home care and other health care facilities as well as hospitals.

3.5 Application illustration

3.6 Application

The Turning Disc is used to aid a transfer.

- Place the Turning Disc under the patient's feet
- The patient's feet must be positioned fully and symmetrically on the Turning Disc.
- To secure and assist the turning movement, the user places one foot on the Turning Disc.
- Depending on the clinical picture, the user supports the patient's forward weight transfer or mobilises patient to standing.
- Additional aids are required/used as necessary.
- Execute the turning movement.
- Sit the patient down.

Visit our website at www.pm-med.de or watch our comprehensive product videos on our YouTube channel.

3.7 Approved accessories, combination with other products

Turning Discs are used in combination with the following products:

- PM-3015 Raising Aid Magic
- PM-3005 Leg Belt
- PM-6013/14/15 Support Belt
- PM-8090 Foot Stool

Petermann transfer aids are part of a comprehensive aid product range, which are effective and back-friendly for the user and comfortable and supportive for the patients. Additional Petermann products may be used according to the situation and clinical picture. Ask our experts. We are happy to help you!

3.7.1 Sets

| Artikel-Nr. Set | Bestandteile Set |
|-----------------|------------------------|
| PM-5085 | PM-5080 Turning Disc |
| | PM-8090 Foot Stool |
| | PM-6014 Support Belt M |
| PM-5085-1 | PM-5080 Turning Disc |
| | PM-8090 Foot Stool |
| | PM-6013 Support Belt S |
| PM-5085-2 | PM-5080 Turning Disc |
| | PM-8090 Foot Stool |
| | PM-6015 Support Belt L |

Please observe the user manuals of the other items as well.

4 Product marking / CE

The Petermann Turning Disc is a Class I Medical Device and meets the EU-Regulation 2017/745 of the European Parliament and Council dated 05.04.2017 for medical devices, also called MDR.

Petermann GmbH is certified in accordance with DIN EN ISO 13485 by TÜV Süd, Medical Devices - Quality management systems applicable to the design and development, production and distribution of aids for people of limited mobility.

5 Cleaning, care and disinfection

| Cleaning | Desinfection (if necessary) |
|--|---|
| Damp cleaning | Recommended disinfectants for wipe disinfection: <ul style="list-style-type: none"> - Trionic® (DGHM/VAH approval, further information on request) - Mikrobac® Tissues* - Mikrobac® forte* - Bacillo® 30 Foam - Bacillo® 30 Tissues - Bacillo® 30 Sensitive Tissues - Mikrobac® Virucidal Tissues* |
| Warm water / mild detergent | |
| Do not use abrasive or bleaching agents! | |
| Do not use sharp-edged objects! | * wipe with a cloth soaked in water after disinfection to avoid residues or layer build-up. Please always follow the manufacturer's manuals for use when wiping disinfection. |

Be sure to follow the care instructions on the label of each product as well as the manuals for use.

6 Technical Data

| Item no. | Material | Colour | Load-bearing capacity |
|----------|--|--------|-----------------------|
| PM-5080 | ABS coated plastic fabric, latex free | beige | 200 kg |

| Item no. | Description | Dimensions in mm | Weight | UDI |
|----------|--------------|---------------------|--------|-------------------------|
| PM-5080 | Turning Disc | Ø 400 mm | 1350 g | (01)04250355350800(10)x |

The manufacturing of our products is subject to highest quality standards. Due to usual manufacturing variations published dimensions and further information concerning the condition of the product are defined as target production specifications. Deviations which might occur are monitored by professional quality management procedures and do not represent a reduction in performance of the respective product.

7 Warranty

- The company Petermann warrants that the product is free from defects in material and workmanship
- The warranty period is 3 years. **It does not cover defects attributable to abrasion, normal wear, improper use or improper cleaning**
- The company Petermann is not liable for accidental or consequential injury or material damage

8 Disposal information

There are no specific disposal regulations for Turning Discs, they can be disposed of in accordance with applicable national and local regulations.

Petermann GmbH
Waldweg 3
D-91601 Dombühl
Phone: +49 (0) 9868-9339-0
Email: petermann@pm-med.de

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