



The Alpha Soft Turning Disc is a simple aid that significantly facilitates lateral movements for seated clients, whether alone or with help. For use in a bed, place the turning disc centrally in the bed approx. 25 cm from the edge of the bed. You thus achieve an optimal lying position in the centre of the bed from the turning movement. While in the seating position on the bed always use the height adjustment of the bed to assist for your own safety. Our Soft Turning Aid is made of breathable cotton.

The Soft Turning Disc can be used for transfers in/out a car seat.  
Warning: Do not leave the product under the user while driving because of the sliding risks in case of an unexpected event or movement.

## Technical specification

Product Name	Material	Colour	Max Lifting weight	Guarantee	Storage/transport
Soft Turning Disc	Cotton, natural thickening, latex free	Cream	200 kg	3 years	5 °C – 40 °C protect against sunlight

Item no.	Dimensions in cm	Weight	UDI
PM-5070	Ø 46	260 g	(01)04250355350701(10)LOT

### Hygiene and maintenance

The product doesn't require any special maintenance. It can be machine washed at 60°C and dried at low temperatures (45° – 50°C). We recommend washing with mild detergents. The use of cleaning agents with bleaching ingredients or other chemical laundry additives, also as fabric softeners may lead to permanent deterioration of the product properties.

